


Body Part Examined

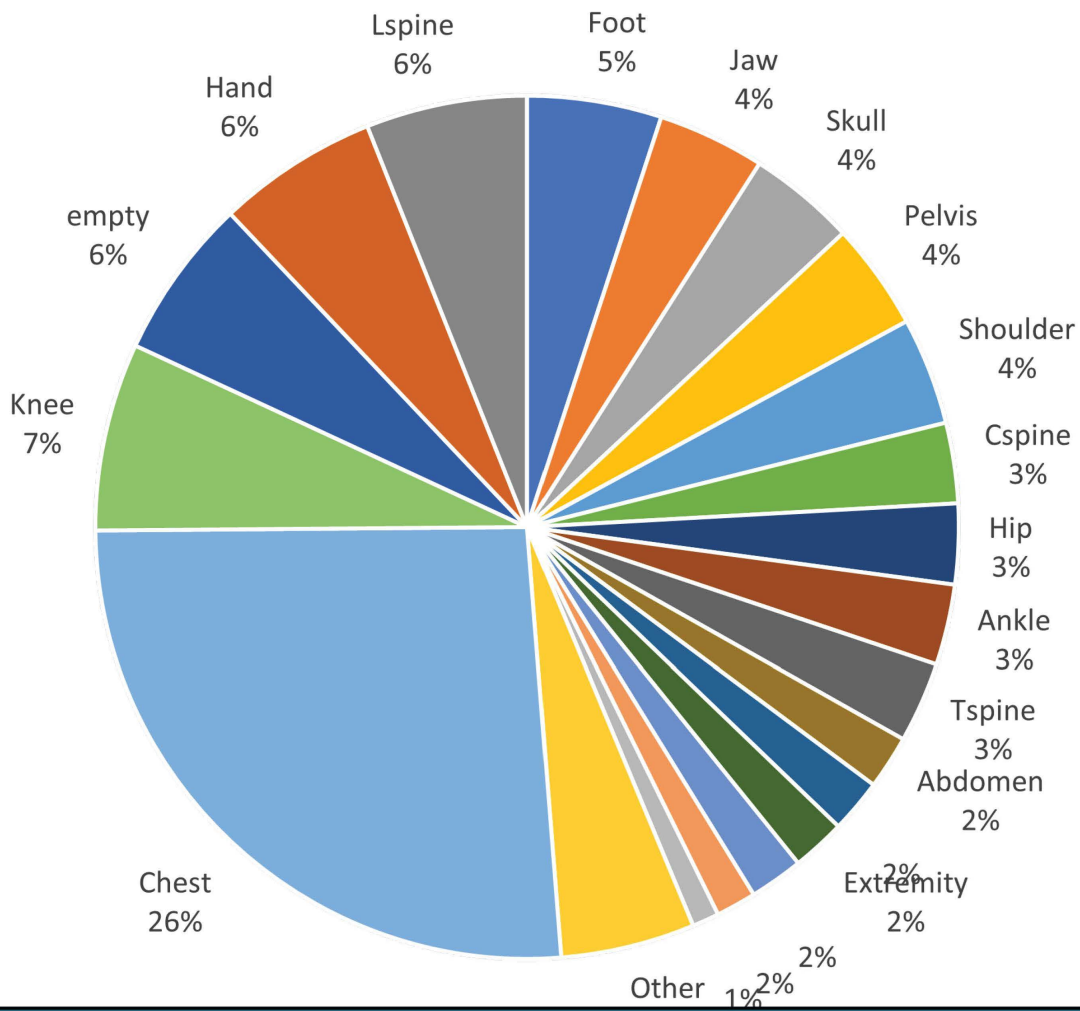

▼ Anon, 02230799133... 02230799133006... Feb-08-2022 12:15 PM CT CHEST W IV CONTRAST CT 36702265689920... 1008

Basic Viewer →

Description	Series	Modality	Instances
(empty)	2	CT	2
Axial Body Std. Axial CE	4	CT	121
Axial Lung Std. Axial CE	5		
Coronal Body Std. Volume CE Coronal	7		
Sagittal Body Std. Volume CE Sagittal	8		
Ax-MIP CTA Body CE Ax-MIP	10		
Cor-MIP CTA Body CE Cor-MIP	11		
Sag-MIP CTA Body CE Sag-MIP	12		

PRE-INDEX

Series lack clinically relevant descriptions

Difficult to distinguish series apart from other body parts

▼ Anon, 02230799133... 02230799133006... Feb-08-2022 12:15 PM CT CHEST SINGLE CT ST688892670104... 1008

Basic Viewer →

Description	Series	Modality	Instances
LOCALIZER	2	CT	2
CHEST AXIAL C+ THICK	4		
CHEST AXIAL C+ LUNG THIN	5		
CHEST CORONAL C+ THICK	7		
CHEST SAGITTAL C+ THICK	8		
CHEST AXIAL C+ MIP	10		
CHEST CORONAL C+ MIP	11		
CHEST SAGITTAL C+ MIP	12	CT	73

POST-INDEX

Empty fields are now identified

Series all have body part included to distinguish from other studies

Pre and Post Index Series Description Mapping

▼

Anon, 02230799133...

02230799133006...

Feb-08-2022

12:15 PM

CT CHEST W IV CONTRAST

CT

36702265689920...

1008

Basic Viewer →

Description	Series	Modality	Instances
(empty)	2	CT	2
Axial Body Std. Axial CE	4	CT	121
Axial Lung Std. Axial CE	5	CT	361
Coronal Body Std. Volume CE Coronal	7	CT	92
Sagittal Body Std. Volume CE Sagittal	8	CT	120
Ax-MIP CTA Body CE Ax-MIP	10	CT	181
Cor-MIP CTA Body CE Cor-MIP	11	CT	58
Sag-MIP CTA Body CE Sag-MIP	12	CT	73

▼

Anon, 02230799133...

02230799133006...

Feb-08-2022

12:15 PM

CT CHEST SINGLE


CT


ST688892670104...

1008

Basic Viewer →


Description	Series	Modality	Instances
LOCALIZER	2	CT	2
CHEST AXIAL C+ THICK	4	CT	121
CHEST AXIAL C+ LUNG THIN	5	CT	361
CHEST CORONAL C+ THICK	7	CT	92
CHEST SAGITTAL C+ THICK	8	CT	120
CHEST AXIAL C+ MIP	10	CT	181
CHEST CORONAL C+ MIP	11	CT	58
CHEST SAGITTAL C+ MIP	12	CT	73

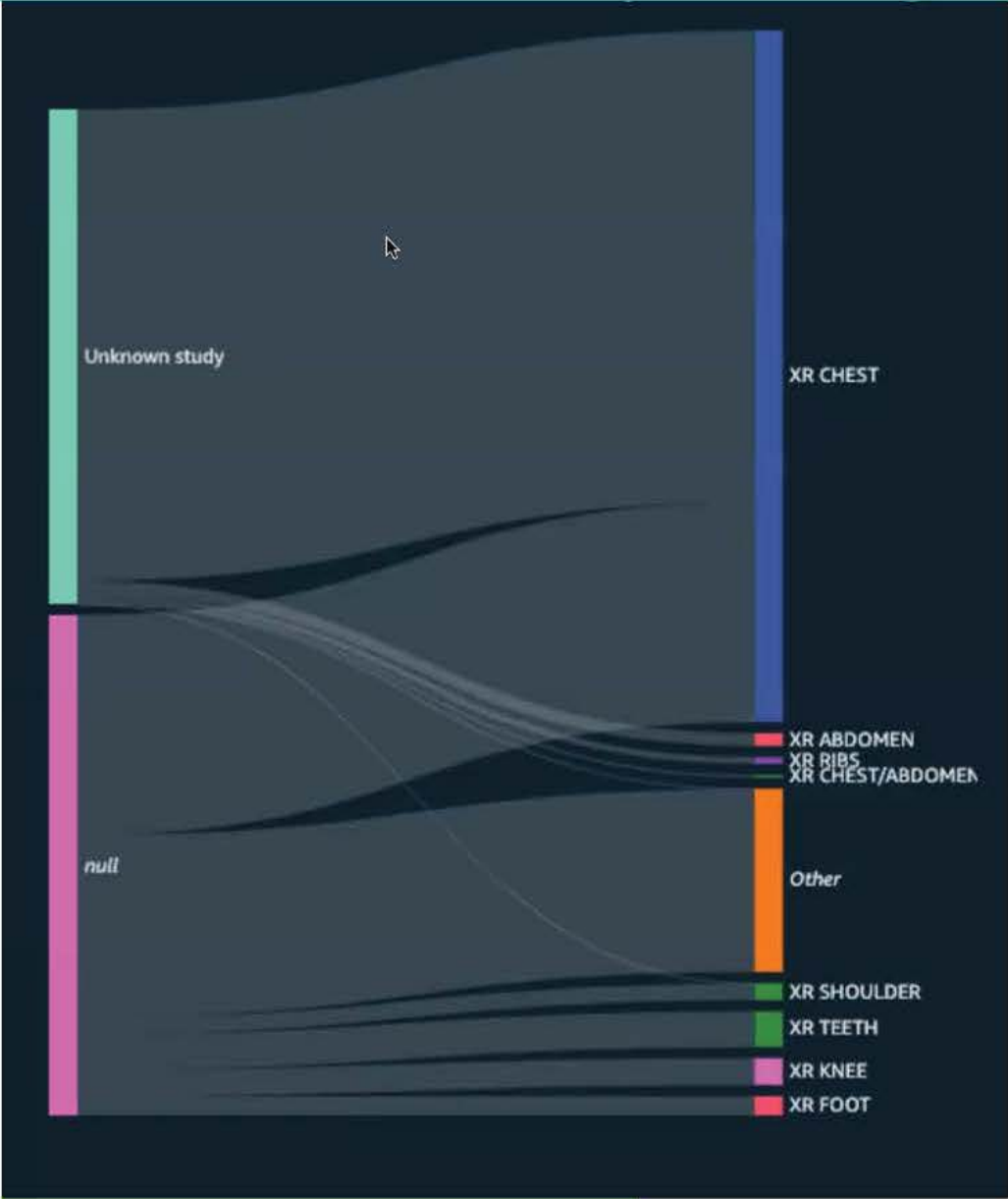

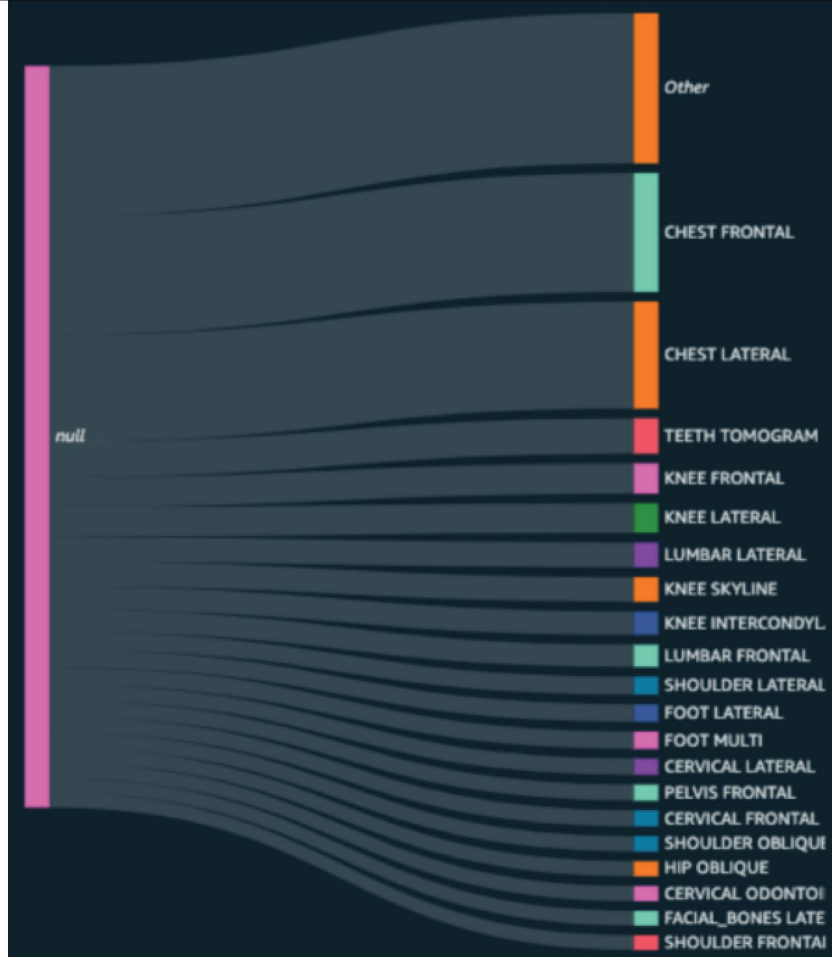

PRE-ENDEX


Multiple series with little clinical relevance are analyzed

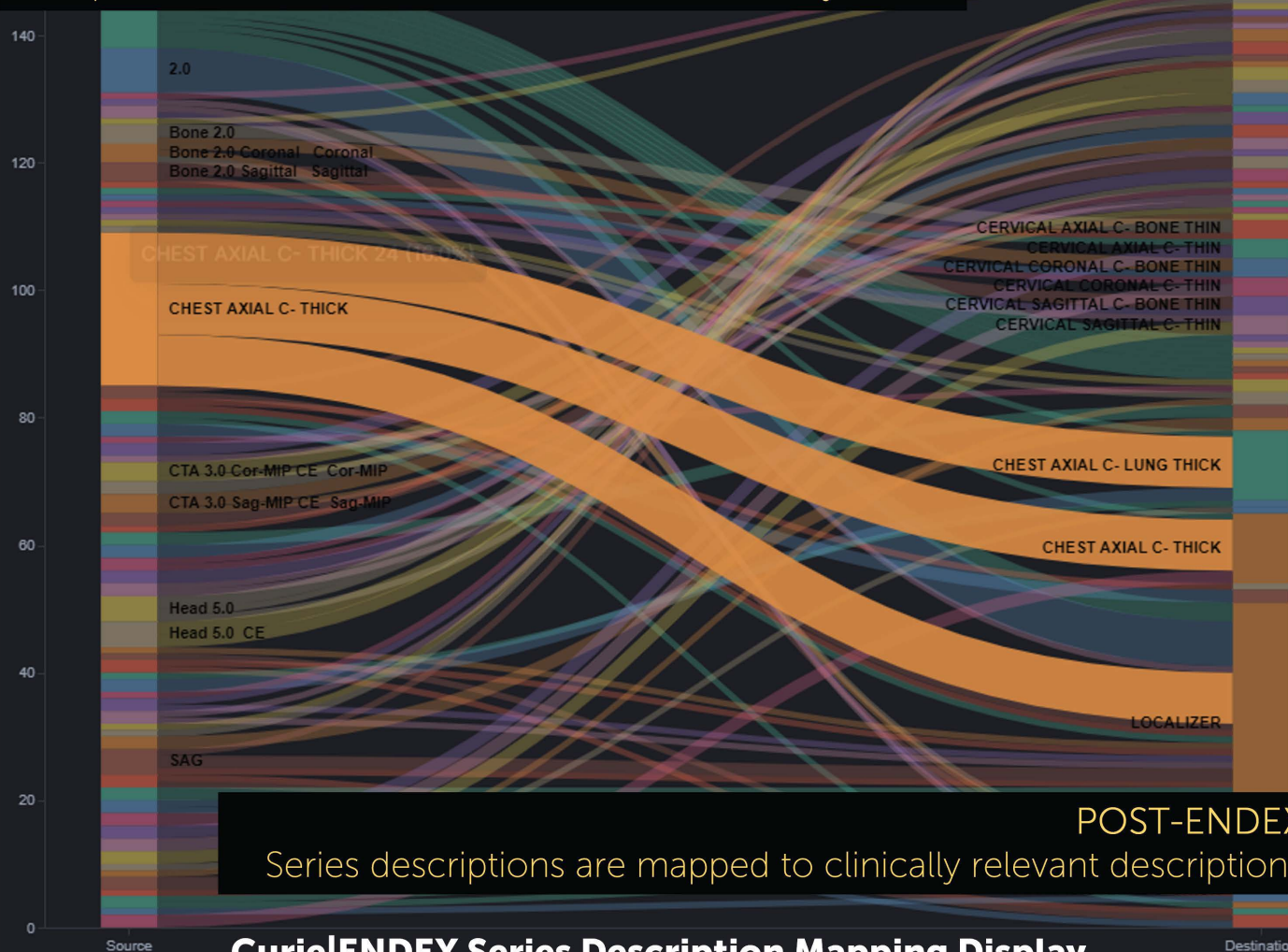

POST-ENDEX

Series descriptions are mapped to clinically relevant descriptions

[SERIES] Original → Standardized (CT)

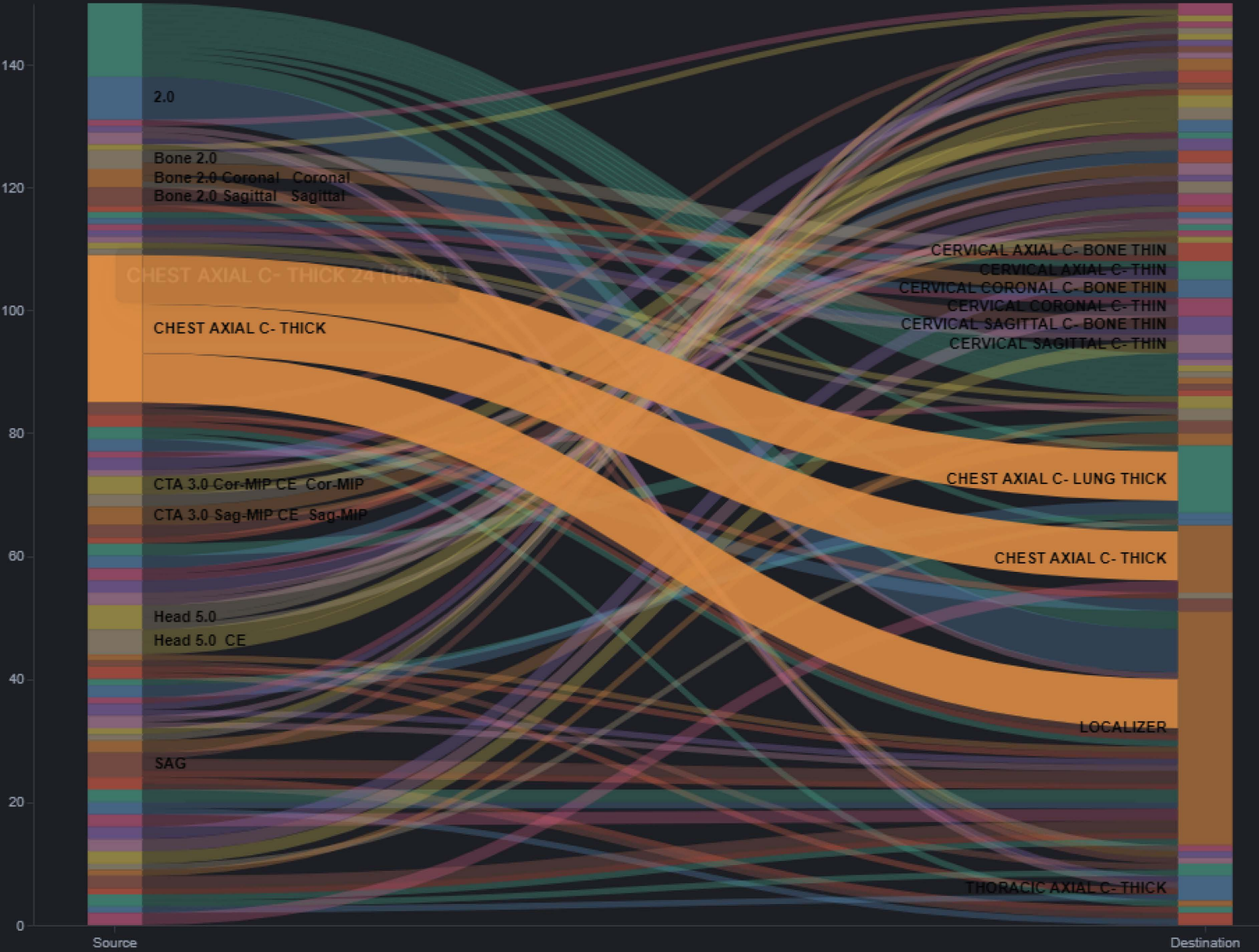

PRE-ENDEX

Multiple series are all named differently despite similar characteristics

Modality	Acquired Series Name	Standard Series Name
CT	ABD PELVIS W/O Coronal.1/Coronal/	ABD PELVIS W/O CONTRAST Coronal
CT	ABD PELVIS W/O Coronal.10/Coronal/	ABD PELVIS W/O CONTRAST Coronal
CT	ABD PELVIS W/O Coronal.109/Coronal/	ABD PELVIS W/O CONTRAST Coronal
CT	ABD PELVIS W/O Coronal.11/Coronal/	ABD PELVIS W/O CONTRAST Coronal
CT	ABD PELVIS W/O Coronal.12/Coronal/	ABD PELVIS W/O CONTRAST Coronal
CT	ABD PELVIS W/O Coronal.13/Coronal/	ABD PELVIS W/O CONTRAST Coronal
CT	ABD PELVIS W/O Coronal.133/Coronal/	ABD PELVIS W/O CONTRAST Coronal
CT	ABD PELVIS W/O Coronal.14/Coronal/	ABD PELVIS W/O CONTRAST Coronal
CT	ABD PELVIS W/O Coronal.143/Coronal/	ABD PELVIS W/O CONTRAST Coronal
CT	ABD PELVIS W/O Coronal.15/Coronal/	ABD PELVIS W/O CONTRAST Coronal
CT	ABD PELVIS W/O Coronal.15/Coronal/	ABD PELVIS W/O CONTRAST Coronal
CT	ABD PELVIS W/O Coronal.15/Coronal/CE	ABD PELVIS W/O CONTRAST Coronal


POST-ENDEX

Series descriptions reflect accurate descriptions of the series enabling better display with hanging protocols

Curie|ENDEX Series Description Mapping Display

Modality	Acquired Series Name	Standard Series Name
CT	ABD PELVIS W/O Coronal.1/Coronal/	ABD PELVIS W/O CONTRAST Coronal
CT	ABD PELVIS W/O Coronal.10/Coronal/	ABD PELVIS W/O CONTRAST Coronal
CT	ABD PELVIS W/O Coronal.109/Coronal/	ABD PELVIS W/O CONTRAST Coronal
CT	ABD PELVIS W/O Coronal.11/Coronal/	ABD PELVIS W/O CONTRAST Coronal
CT	ABD PELVIS W/O Coronal.12/Coronal/	ABD PELVIS W/O CONTRAST Coronal
CT	ABD PELVIS W/O Coronal.13/Coronal/	ABD PELVIS W/O CONTRAST Coronal
CT	ABD PELVIS W/O Coronal.133/Coronal/	ABD PELVIS W/O CONTRAST Coronal
CT	ABD PELVIS W/O Coronal.14/Coronal/	ABD PELVIS W/O CONTRAST Coronal
CT	ABD PELVIS W/O Coronal.143/Coronal/	ABD PELVIS W/O CONTRAST Coronal
CT	ABD PELVIS W/O Coronal.15/Coronal/	ABD PELVIS W/O CONTRAST Coronal
CT	ABD PELVIS W/O Coronal.15/Coronal/	ABD PELVIS W/O CONTRAST Coronal
CT	ABD PELVIS W/O Coronal.15/Coronal/CE	ABD PELVIS W/O CONTRAST Coronal

85% of Series Descriptions Normalized by Index

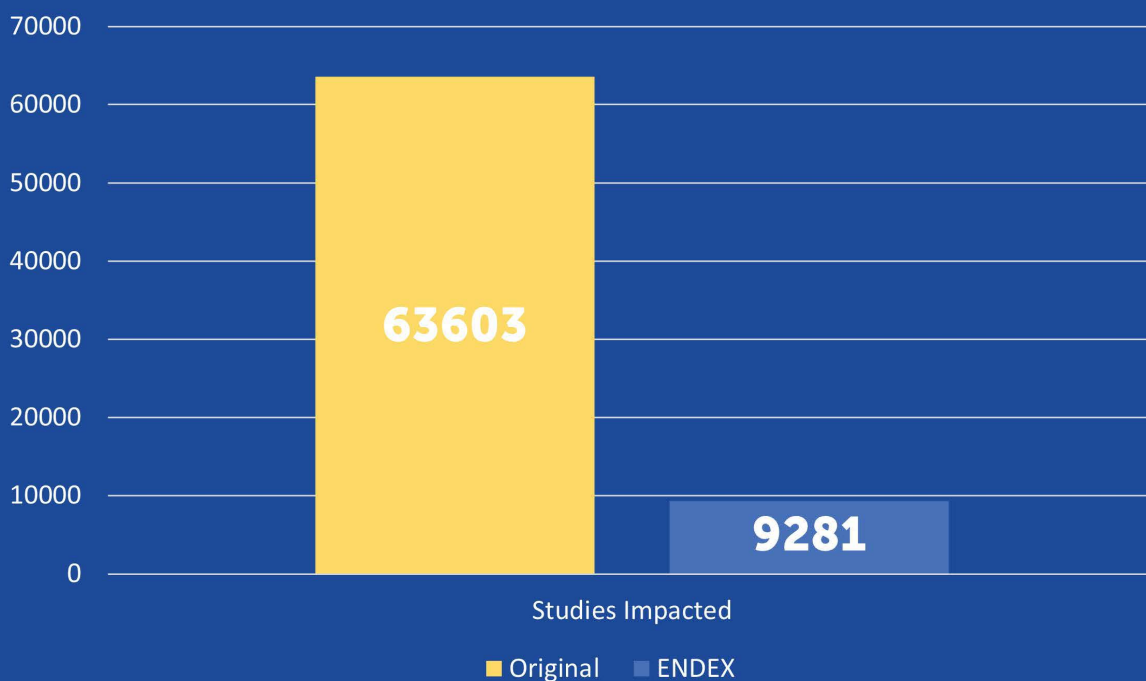

22% of Study Descriptions Normalized by Index

